


Eylea 8mg solution for injection in pre-filled syringe

The pre-filled syringe with OcuClick dosing system is for single use in one eye only. Extraction of multiple doses from a single pre-filled syringe with OcuClick dosing system may increase the risk of contamination and subsequent infection.




Do not use if the package or its components are expired, damaged, or have been tampered with. Check the label on the pre-filled syringe with OcuClick dosing system to make sure you have the strength of Eylea that you intended to use. The 8 mg dose requires use of the Eylea 114.3 mg/ml pre-filled syringe.

The intravitreal injection should be performed with a 30 G × ½ inch injection needle (not included). Use of a smaller size needle (higher gauge) than the recommended 30 G × ½ inch injection needle may result in increased injection forces.

Pre-filled syringe with integrated OcuClick dosing system description

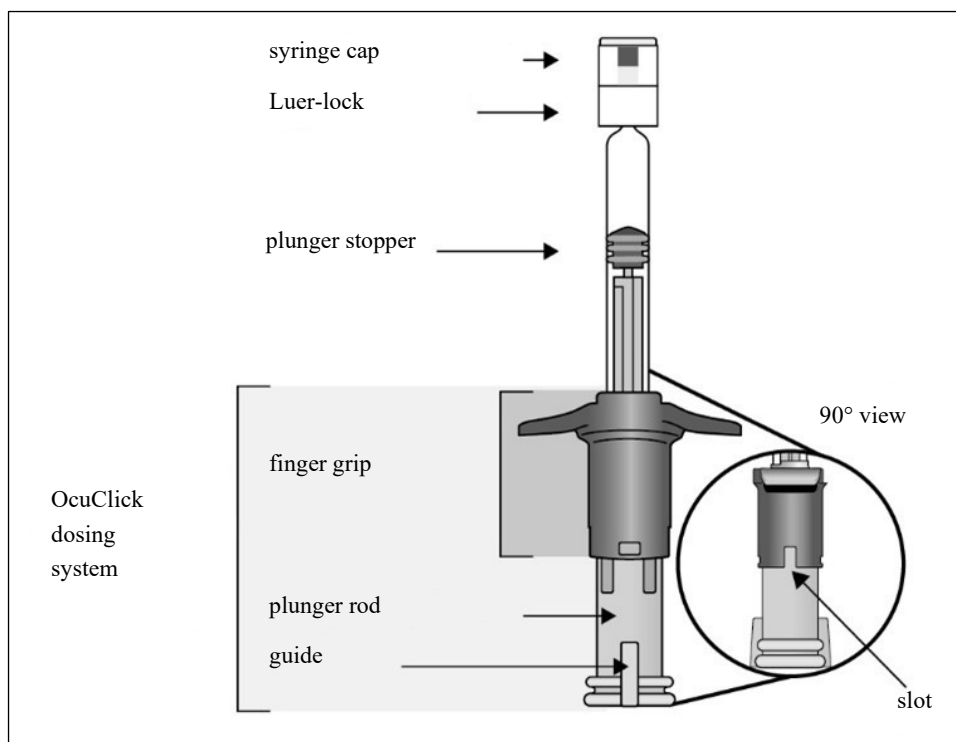

| | |
|----|---|
| 1. | <p>Prepare</p> <p>When ready to administer Eylea 114.3mg/ml, open the carton and remove the sterile blister. Carefully peel open the blister ensuring the sterility of its contents. Keep the syringe in the sterile tray until you are ready to attach the injection needle.</p> <p>Use aseptic technique to carry out steps 2-9.</p> |
| 2. | <p>Remove syringe</p> <p>Remove the syringe from the sterilised blister.</p> |
| 3. | <p>Inspect syringe and solution for injection</p> <p>Do not use the pre-filled syringe if</p> <ul style="list-style-type: none">- particulates, cloudiness, or discolouration are visible- any part of the pre-filled syringe with OcuClick dosing system is damaged or loose- the syringe cap is detached from the Luer-lock. |

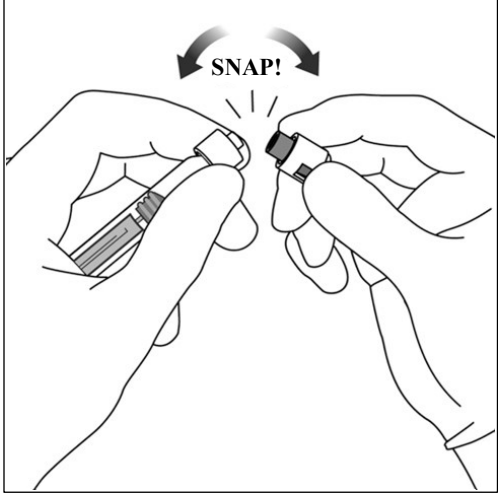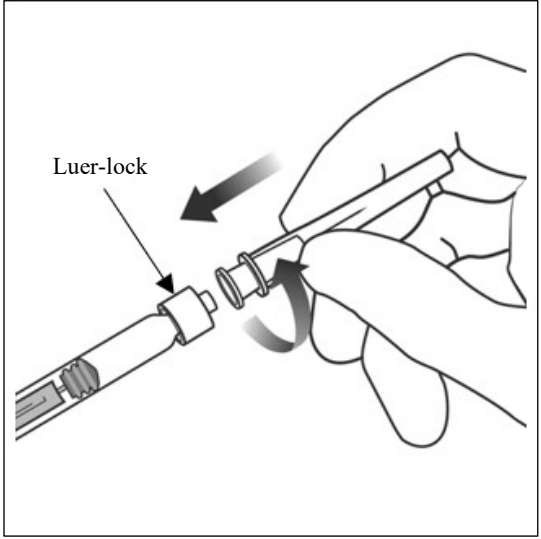
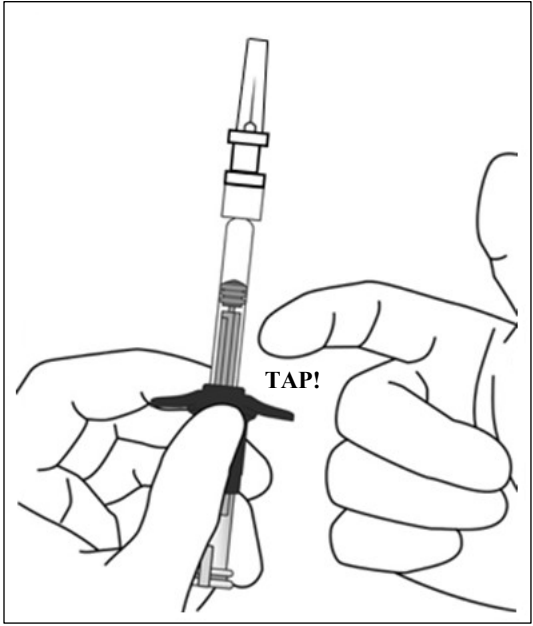
| | | |
|-----------|--|--|
| <p>4.</p> | <p>Snap off syringe cap</p> <p>To snap off (do not twist off) the syringe cap, hold the syringe in one hand and the syringe cap with the thumb and forefinger of the other hand.</p> <p>Note: Do not pull back on the plunger rod.</p> |  |
| <p>5.</p> | <p>Attach needle</p> <p>Firmly twist the 30 G × ½ inch injection needle onto the Luer-lock syringe tip.</p> |  |
| <p>6.</p> | <p>Dislodge air bubbles</p> <p>Holding the syringe with the needle pointing up, check the syringe for bubbles. If there are bubbles, gently tap the syringe with your finger until the bubbles rise to the top.</p> |  |

7. Expel air and excess volume to prime

The syringe does not have a dose line because it is designed to set the dose mechanically as explained in the steps below.

Priming and setting the dose must be done using the following steps.


To eliminate all bubbles and to expel excess medicinal product, slowly depress the plunger rod (left picture below) until it stops, i.e. when the guide on the plunger rod reaches the finger grip (right picture below).

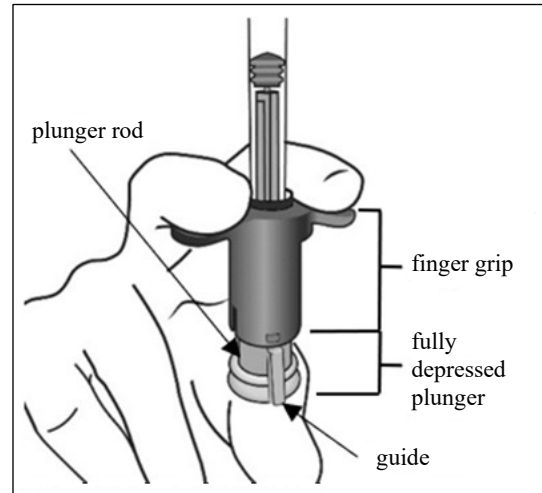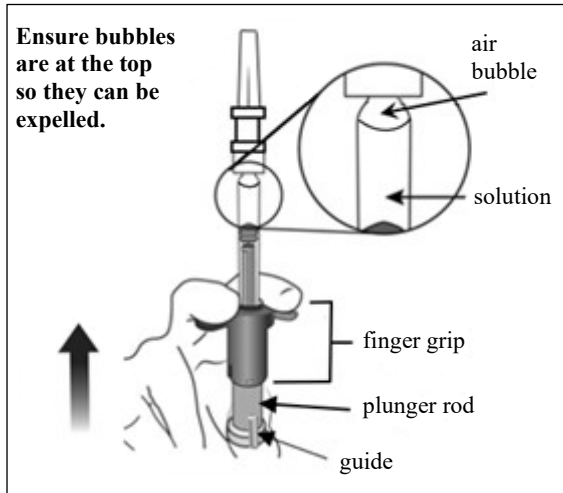


8. Set the dose

Turn the end of the plunger rod 90 degrees clockwise or counter clockwise until the guide of the plunger rod aligns with the slot. You may hear a 'click'.

Note: Now the device is ready to dose. Do not push the plunger rod before insertion into the eye.

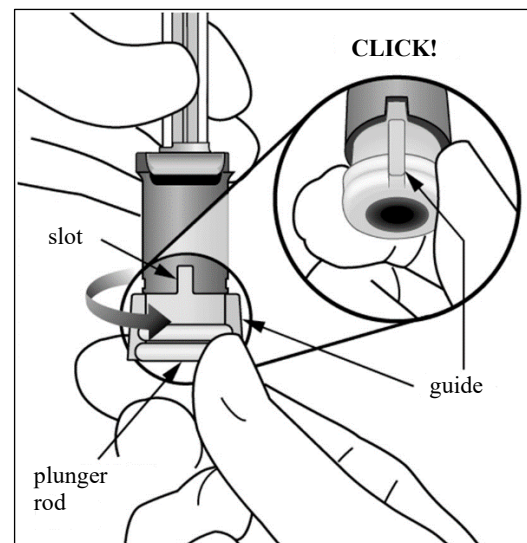
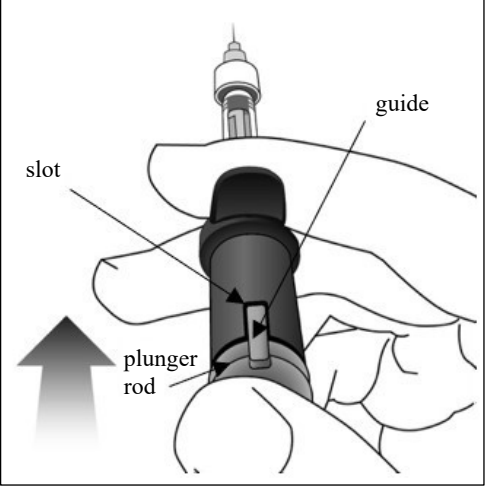
| | | |
|-----|---|--|
| 9. | <p>Administer the injection</p> <p>Insert the needle into the ocular injection site. Inject the solution by pushing in the plunger rod until it stops, i.e. until the guide is completely within the slot.</p> <p>Do not apply additional pressure once the guide is within the slot. It is normal to see a small amount of residual solution left in the syringe.</p> |  |
| 10. | <p>The pre-filled syringe is for single dose administration and single use only. After injection discard the used syringe into a sharps container.</p> | |

Any unused medicinal product or waste material should be disposed of in accordance with local requirements.